
**CARDIAC
THERMOWRAP**


**UNIVERSAL
THERMOWRAP**

**INFANT
THERMOWRAP**


1

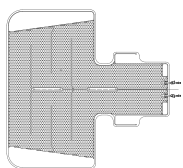
1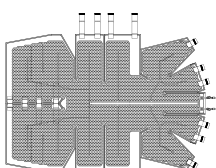

1

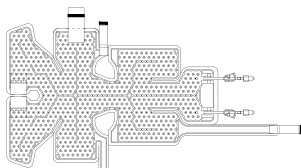2

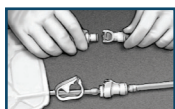
2


2


3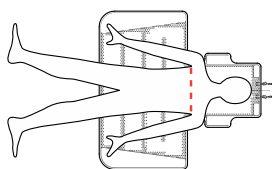


3

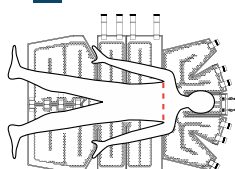3

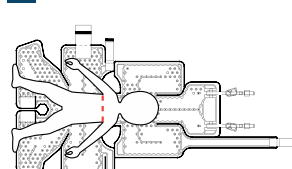4-1

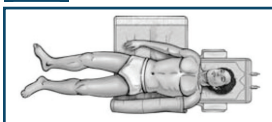
4-1

4-1

4-2

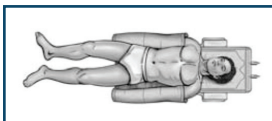
4-2

4-2

4-3

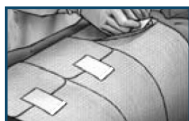4-3

- 1** Unroll the ThermoWrap on the bed with the tubes towards the head of the bed with the lot and part numbers legible.

NOTE: ThermoWrap does not replace standard cushioned mattresses.

- 2** A. Connect ThermoWrap to Allon by inserting the wrap tubes into the plastic ends of the blue Connecting Water Hoses, listening for the click.
B. Ensure that the clamps on the ThermoWrap are open.
C. Activate Allon.

NOTE: In case of leakage, Allon should be shut down.

- 3** D. Position patient on ThermoWrap with the patient's head towards the tubes.
E. Refer to the diagram for correct armpit positioning.

CAUTION: Do not use ThermoWrap to lift patient. Make sure ThermoWrap is not folded or clamped.

- 4** A. Wait until ThermoWrap fills with water prior to securing it around the patient.
B. When closing the ThermoWrap around the patient, ensure that it fits loosely. (There should be a finger's width space between the patient and the ThermoWrap).

WARNING: Prevent any thermal isolation, such as a pillow or other items, between the Wrap and the patient's body.

CAUTION:

- U.S. Federal Law restricts this device for sale to or on the order of a physician.
- If treatment continues longer than 28 hours, replace ThermoWrap.
- Avoid handling ThermoWrap with sharp objects.

TO DISCONNECT THERMOWRAP:

- 5** A. Shut down Allon or put on Standby Mode.
B. Wait a few seconds for water to leave the wrap.
C. Close clamps on the wrap and disconnect.
D. Carefully remove ThermoWrap from patient.

Intended use – ThermoWrap is a single use device designed exclusively for use with the Allon 2001 System. ThermoWrap is intended to maintain preset body temperature as determined by a physician when used with the Allon 2001. It can be utilized to maintain normal body temperature during surgical procedures. See Allon 2001 user manual for a full list of warnings / contraindications.

Patient Target Group – ThermoWrap can be used with adult and pediatric patients.

Intended Users – Trained health care professionals, including nurses and physicians.

Contraindications – ThermoWrap should not be in direct contact with open, widespread skin lesions such as burns or dermatitis. Caution should be taken when using ThermoWrap with patients that have underlying skin conditions.

Residual Risk – Possible patient injury could occur if the garment is wrapped around the patient too tightly.

Device Disposal – ThermoWrap garments are designated as general medical waste. ThermoWrap should be disposed of in the general waste stream.

Risk of Reuse – ThermoWrap is for single use only. Reuse could lead to cross contamination and/or irritation.

Note: Any serious incident that occurs in relation to this device should be reported to Belmont Medical Technologies and to the competent authority of the member state in which the user and/or patient is established.

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